Rocanal Permanent Gangrene R3

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
Rocanal Permanent Gangrene R3

A permanent root canal filling cement for infected root canals

COMPOSITION

Liquid (Alcoholic solution of Eugenol): Eugenol, Rosin, Castor oil, Glycerol, Alcohol	Powder Zinc oxide, Titanium oxide, Zinc stearate, Zinc acetate, Orthophenylphenol (OPP), Calcium wolframate, Anise
--	--

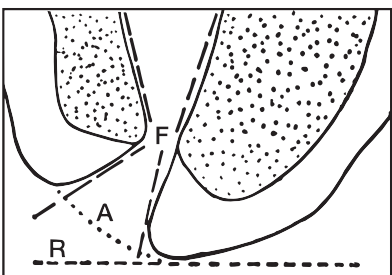to prepare just before use (see mixing instructions hereafter)

F = physiological foramen
A = anatomical foramen
R = radiological foramen

Root canal filling after treatment of a non vital tooth
(reaches the anatomical foramen or goes slightly over)

GB

INDICATIONS

A permanent sealing of infected root canals to be used after complete preparation of gangrenous canals, for adolescents over 12 years of age (at least 30 kg) and adults.
It is advisable to treat one tooth per day with a maximum of 5 treatments per year for adolescents over 12 years and 10 treatments per year for adults.

CONTRAINDICATIONS / SIDE EFFECTS

Not indicated in case of intolerance or allergy to one of the components and in particular in case of allergy or suspected allergy to Eugenol. A temporary periapical parodontitis may appear after applying the product and resolves in two days with proper treatment. No side effects have been reported to date.

DEFINITION / REMARKS:

The permanent root canal filling is the final step in the treatment of a gangrenous tooth.
The gangrenous tooth, on the contrary of a vital one, has the root canal completely infected up to the apical region. The defense against this bacterial action is in the well irrigated periodontium. In the canal the natural barrier is the limit between the enamel and the primary cement.

Diagnosis: X-ray-Symptoms -objective examination
Asymptomatic (chronic) cases: tooth is necrotic -presence of large fillings -fistula -history of an acute anterior episode -negative response to pulptest; etc.
Acute cases: periodontal pain -periapical abscess, periostitis, sub mucus abscess etc.

GENERAL PRINCIPLES

- Isolate the tooth with a coffer dam.
 - The root canal should never be left open.
 - Avoid partial root canal preparations and partial intracanal medications, because it may easily cause infection or acute periapical reaction. Canal treatment and cleaning must be completed in the same session.
 - Control always if there are deep gingival pockets.
- The difference between the treatment of a vital tooth and a non vital one, is that for the first the operator follows the surgical principle of "open and close immediately" in the same session if possible. For the non vital/infectd case, it is necessary to proceed with a particular preparation of both the canal and the periapical region before performing the permanent filling of the canal (which will be performed in a subsequent session).

TREATMENT SYSTEM

First session

1. Immediate and complete cleaning of the infected canal as far as the apex, by means of mechanical and chemical action (antiseptic). It is recommended to thoroughly irrigate with ROCANAL IRRIGATION, rinse and dry.
2. Preventive drug prescription for the periapical region if necessary.

Second session

Before filling permanently, it is essential to shape, clean and lubricate with ROCANAL IRRIGATION or the product of your choice. Final root-canal filling as far as the apex (anatomic foramen)

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REV. :10/2020

CE
0476

use ROCANAL PERMANENT GANGRENE R3.
ROCANAL PERMANENT GANGRENE R3 has especially been developed to meet the requirements of a filling material to be used after treatment of non vital teeth.

Action of ROCANAL PERMANENT GANGRENE

Action studied in function of unextracted and infected pulp remnants.

ROCANAL PERMANENT GANGRENE is active over two distinct phases: the soft phase and after hardening. It is not reabsorbed in the canal system.

PRODUCT INSTRUCTIONS:

1. Use sterile gloves and a sterile mixing slab and two single-end flexible sterile spatulas (one for powder, one for mixing).
2. Put one or two drops of liquid and small quantity of powder on the slab.
3. Amalgamate powder slowly with the liquid, with some pressure on the spatula.
4. Mix as long as needed to obtain a creamy, crystal-free mixture (crystals present in the powder are soluble in the liquid).
5. Proceed with your usual filling technique.
6. Use sterile material.
7. The setting time will depend on the consistency desired by the practitioner, depending on the powder/liquid ratio and the temperature.
The average time of hardening of the mixture is 3 to 4 minutes.

PRECAUTIONS FOR USE

For professional use only.
Eliminate balances of unused product with the official rules in force.

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