



MAINTENANCE OF TURBINES.

Refer to the user manual for the product for the complete cleaning and sterilisation maintenance procedures. This medical device is supplied in "non sterile" condition. Please ensure that this procedure is followed before the device is used.

1 PREPARATION



A. REMOVING THE INSTRUMENT

- Remove the rotary instrument from the chuck mechanism

B. PRE-DISINFECTION

- External disinfection

⊘ Do not immerse the instrument in a disinfectant bath.

2 CLEANING



A. CLEAN UNDER RUNNING WATER AT <38°C

B. REMOVING DIRT/DEPOSITS

- Carefully remove all traces of dirt or deposits from the spray nozzles using the Bien-Air cleaning wire
- Dry the nozzles with compressed air then dry the rest of the instrument with a cloth


⊘ Do not immerse in an ultrasonic cleaner



C. INTERNAL AND EXTERNAL CLEANING WITH SPRAYNET

- Shake the **Spraynet** before use
- Use the **Spraynet** attachment ①* (for 2-/3-way and 4-way turbines, see inset below)
- Keep the tip of the instrument pointing down and protect with an absorbent cloth; press the **Spraynet** button down for 1 second
- Clean the exterior of the turbine with a cloth soaked in **Spraynet**

FOR 2-/3-WAY AND 4-WAY TURBINES ONLY

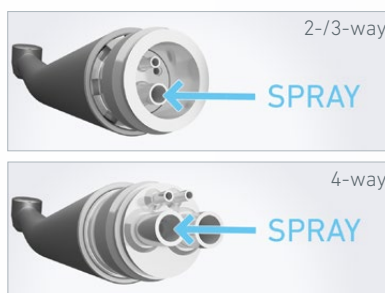

C'. CLEANING THE INSIDE WITH SPRAYNET FOR 2-/3-WAY AND 4-WAY TURBINES ONLY

- Use the **Spraynet** attachment ②*
- Keep the tip of the instrument pointing down and protect with an absorbent cloth; press the **Spraynet** button down for 1 second

△ Spray into the opening as shown in the illustrations for "2-/3-way" and "4-way"

3

DISINFECTION



A. DISINFECTION USING SURFACE DISINFECTANT
(follow the manufacturer's instructions for use)



⊘ Do not immerse the instrument in a disinfectant bath

OR

B. THERMAL DISINFECTION
(follow the manufacturer's instructions for use)

4

LUBRICATION

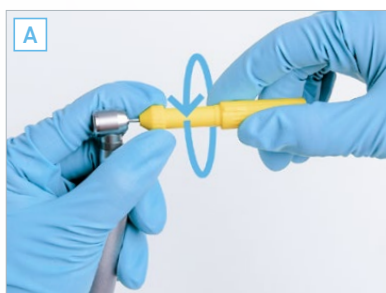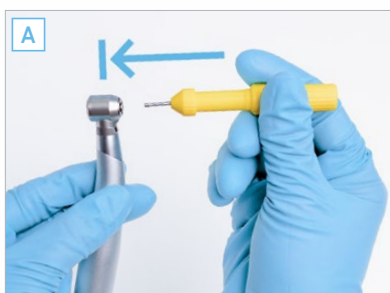

A. LUBRICATION WITH LUBRIMED

OR

B. LUBRICATION WITH LUBRIFLUID

- Shake the Lubrifluid before use
- Use the Lubrifluid attachment ②, ③ or ④ depending on the type of instrument*
(For 2-/3-way and 4-way turbines, see inset below)


- Keep the tip of the instrument pointing down and protect with an absorbent cloth; press the Lubrifluid button down for 1 second

- Then, run an operating test for 30 seconds at low speed to remove any surplus lubricant

- Store immediately: store the instruments with the tip pointing downwards so that any oil residues can flow out



FOR 2-/3-WAY AND 4-WAY TURBINES ONLY

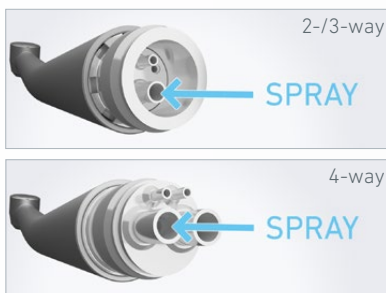

B'. INTERNAL LUBRICATION WITH LUBRIFLUID FOR 2-/3-WAY AND 4-WAY TURBINES ONLY

- Use the Lubrifluid attachment ②*
- Keep the tip of the instrument pointing down and protect with an absorbent cloth; press the Lubrifluid button down for 1 second

⚠ Spray into the opening as shown in the illustrations for "2-/3-way" and "4-way"

5

STERILISATION



A. PLACE THE TURBINE IN A STERILISATION BAG

B. STERILISE IN ACCORDANCE WITH THE MANUFACTURER'S INSTRUCTIONS

- Instruments bearing the 135°C symbol can be sterilised in an autoclave

⚠ Only sterilise instruments which have been cleaned and lubricated

* All the references for the attachments are given on the last page of the document.

MAINTENANCE

ACCESSORIES



SPRAYNET
500 ml cleaning spray
Pack of 6
REF. 1600036-006



Bora L / LK, Boralina,
Prestige L / LK, Tornado L / LK



Black Pearl Eco 3- and 4-way



LUBRIFIUID
500 ml lubricant
Pack of 6
REF. 1600064-006



Black Pearl Eco 3- and 4-way



Bora L, Boralina, Prestige L,
Tornado L, Boralina



Bora LK, Prestige LK, Tornado LK



AQUACARE
500 ml cleaning spray
Pack of 6
REF. 1600617-006



LUBRICARE
Maintenance system
REF. 1600915-001



LUBRIMED
Pack of 6 cartridges
REF. 1600037-006



BIEN-AIR CLEANING WIRE
Pack of 10
REF. 1000001-010