R-100210

ES.....	DESTILADOR / DESCALCIFICADOR.....	4
EN	DISTILLER / DECALCIFIER.....	6
FR.....	MACHINE À DISTILLER ET À DÉCALCIFIER	8
PO	DESTILADOR / DESCALCIFICADOR.....	10
IT.....	DISTILLATORE DECALCIFICATORE	12

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TALLERES MESTRAITUA S.L.
Txori-Erri Etorbidea, 60
Tfno. + 34 944530388 - Fax + 34 944711725
E-mail: mestra@mestra.es - www.mestra.es
48150 SONDIKA - BILBAO - ESPAÑA

Rev. 21/09/15

Contenido del embalaje / Contents of the packing / Contenu de l'emballage
Conteúdo da embalagem / Contenuto dell'imballaggio




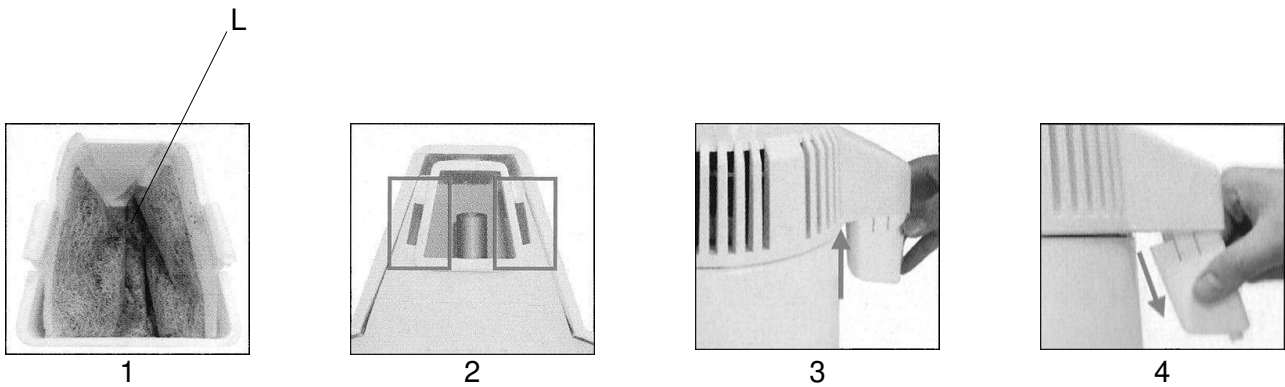
Fig. 1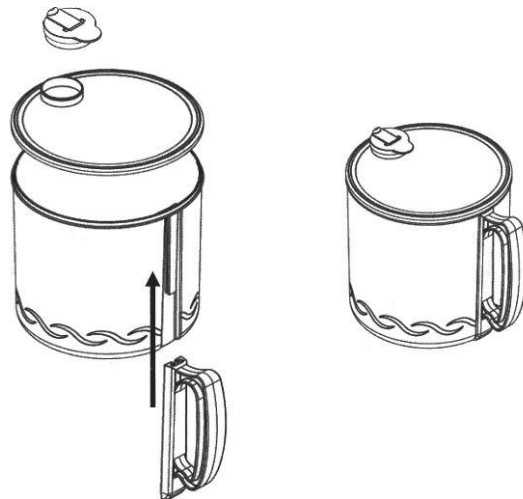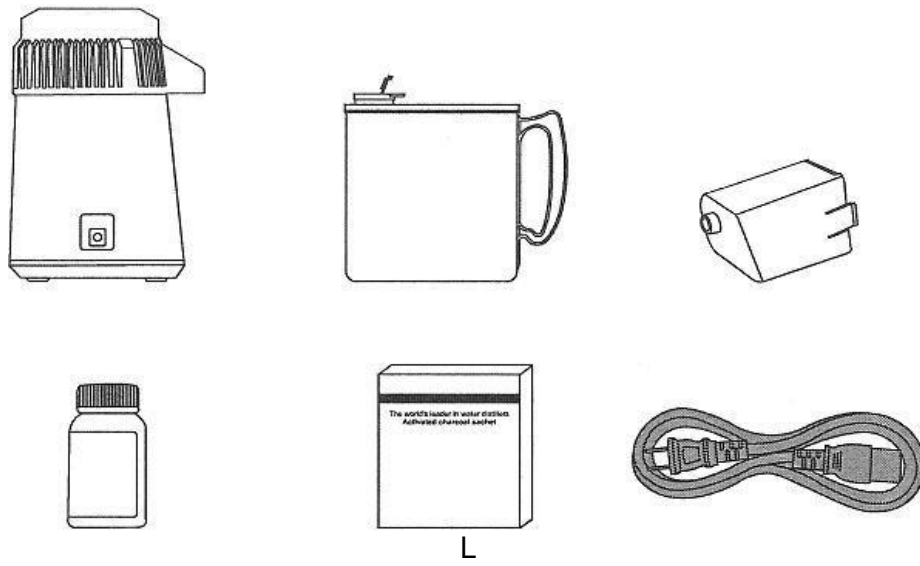

- | | |
|---|--|
| A | Soporte tapa / Lid holder / <i>Support couvercle</i> / Suporte tampa / Supporto coperchio |
| B | Junta / O-ring / <i>Joint</i> / Junta / Guarnizione |
| C | Calderín / Tank / <i>Réservoir</i> / Reservatório / Riscaldatore |
| D | Interruptor (reset) / Switch (reset) / <i>Interrupteur (reset)</i> / Interruptor (reset) / Interruttore (reset) |
| E | Tapa / Lid / <i>Couvercle</i> / Tampa / Coperchio |
| F | Serpentín / Coil / <i>Serpentin</i> / Serpentino / Serpentino |
| G | Ventilador / Fan / <i>Ventilateur</i> / Ventilador / Ventilatore |
| H | Salida agua / Water outlet / <i>Sortie d'eau</i> / Saída água / Scarico acqua |
| I | Cable ventilador / Fan power cord / <i>Câble ventilateur</i> / Fio ventilador / Cavo ventilatore |
| J | Conexión ventilador / Fan connection / <i>Connexion ventilateur</i> / Ligação ventilador / Collegamento ventilatore |
| K | Conexión alimentación / Power connection / <i>Connexion alimentation</i> / Ligação alimentação / Collegamento alimentazione |
| L | Filtro de carbón activo / Activated carbon filter / <i>Filtre de charbon actif</i> / Filtro de carvão activo / Filtro a carbone attivo |

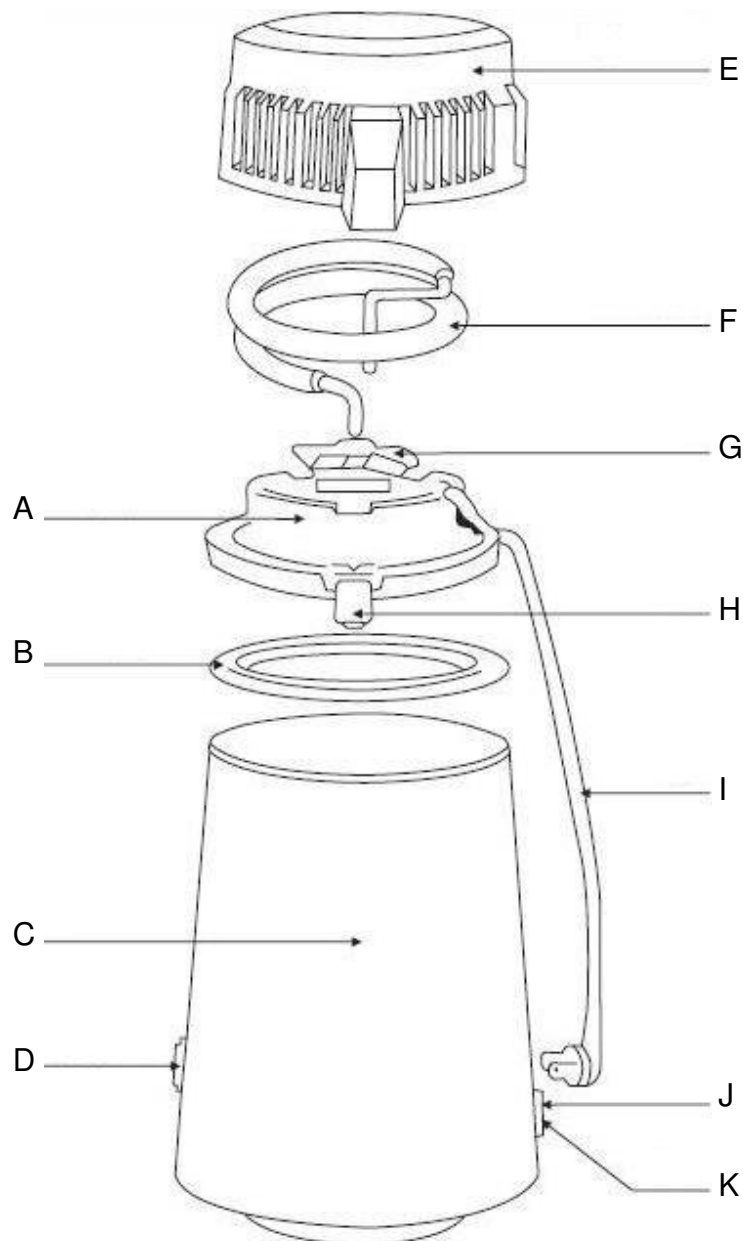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The R-100210 distiller/decalcifier is a simple and practical appliance: you will obtain 4 litres of distilled water by just filling the tank with tap water and waiting for about 6 hours. Its operation is fully automatic and silent. Cleaning the tank is easy and quick. The water produced is very pure (between 2 and 10 mg/litre of solid residue) and can be used in a large number of applications in the clinic and the dental lab (autoclaves, compound making, steam generating machines, etc.).

UNSUITABLE WATER FOR SURGICAL USE!

IMPORTANT- The appliance produces water of the "DISTILLED" type. It cannot be used in injections, vials, and wound cleaning or in any other surgical operation that requires "TRI-DISTILLED" type water.

SAFETY

- Read these instructions carefully before operating the unit.
- Keep these instructions for further consultation.

- Install the distiller on a dry, stable, heat-resistant and well-ventilated surface.
- The distiller requires a well-ventilated room. Bear in mind that while operating it emits heat.
- Use the unit exclusively for the aim it was designed.
- Do not allow children or non-qualified personnel handling the unit.
- Keep the citric acid container out of children's reach.

- Disconnect the unit from the power source for maintenance operations.
- Unplug the distiller for cleaning or refilling it.
- Do not dip any element of the distiller.
- If you need to interrupt the distillation cycle, stop the distiller by unplugging it.

- Do not open the distiller while it is working.
- Wait at least 20 minutes after a distillation cycle to open the distiller for cleaning it.
- Do not fill the tank with boiling water.

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- Do not use cord extensions or connect any other appliance into the same outlet.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents...)

OPERATION

1. Put an activated carbon packet (L) in the nozzle. Rinse and clean it under the tap (Fig 1).

Note: fold the filter by half to place it in the cap.

2. Put the nozzle in its place (Fig 1).
3. Open the lid (E). Note: to do this, do not pull the handle.
4. Pour water in the tank making sure not to exceed the mark. It is about 5 cm from the top.

Note: we recommend using hot water to shorten the time of the cycle.

5. Make sure all the contacts, wires and connections are not wet.
6. The lid of the cap must be in line with the outlet hole of the tank. Plug the fan power cord (K) and the main power cord in (J). You will notice that the fan starts working.
7. Turn the "RESET" switch on to start the distillation cycle.
8. The condensation process starts approximately one hour after the beginning of the cycle.
9. The distiller will disconnect automatically after finishing the cycle.

CLEANING THE TANK

- Fill the tank with 4 litres of hot water.
- Add two spoonfuls of cleaner (too much acid may damage the material of the tank).
- Leave the lid open, connect the power cord in (J) and press the switch (D) to heat the tank until the water boils.
- Empty the tank after 15 or 20 minutes. Then, wash the inside with clean water.
- If there is too much residue to be removed in one go, repeat the process several times until the inside is completely clean.
- A small amount of residue can cause no damage and you can use the unit safely.

DO NOT ADD ACID TO WATER. The acid will not evaporate, will increase the acidity cycle after cycle and the it will finally perforate the tank.

MAINTENANCE

- Do not use abrasive cleaners or brushes. Use only a specific cleaner for distillers.
- Clean the tank approximately once a week or more often if the water is hard.
- Allow enough time for the cleaner to work and dissolve the particles set into the tank.
- The filter pack should be replaced at least once a month.
- If the coil is dirty, the distillation process will be less efficient. Clean the coil regularly through the upper grille with a brush and suction pump.
- After each operation, rinse the tank to prevent the residues hardening.
- Keep the outside of the unit dry.

TECHNICAL SPECIFICATIONS

Voltage:	220 V, 50/60 Hz
Power:	580 W
Resistance:	565 W
Fan:	15 W
Distillating capacity:	approx. 4 L in 6 hrs.
Height:	365 mm
Width:	250 mm
Depth:	280 mm
Tank capacity:	4 l
Weight:	3,5 kg

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