R-100009

E	CALENTADOR DE INSTRUMENTOS POR INDUCCIÓN	4
GB	INDUCTION TOOL HEATER	6
D	INDUKTIONSHHEIZGERÄT FÜR INSTRUMENTE	8
P	AQUECEDOR DE INSTRUMENTOS POR INDUÇÃO	10
I	RISCALDATORE STRUMENTI A INDUZIONE	12

MESTRA[®]

TALLERES MESTRAITUA S.L.

Txori-Erri Etorbidea, 60

48150 SONDIKA - BILBAO - ESPAÑA

Tel. +34 944530388 - Fax +34 944711725

mestra@mestra.es - www.mestra.es

Rev. 09/06/09

R-100009

- A Sensores / **Sensors** / *Sensoren* / Sensores / **Sensori**
- B Cavity de inducción / **Induction cavity** / *Induktionsöffnung* / Cavity de indução / **Cavità d'induzione**
- C Display / **Display** / *Display* / Display / **Display**
- D Ajuste sensibilidad / **Sensitivity adjustment** / *Empfindlichkeitseinstellung* / Ajuste sensibilidade / **Regolazione sensibilità**
- E Protector desechable / **Disposable shield** / *Wegwerfbare Schutzkappe* / Protector descartável / **Protettore usa e getta**

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- Carefully read this manual. It contains important information on safety during the installation, operation and maintenance of the unit.
- Keep this manual in a suitable place to consult it in the future.
- Check the overall state of the unit just upon unpacking it.
- Before connecting the unit, make sure the voltage shown in the identification plate matches that of the power source to be used. The installation must fulfil all the applicable safety standards and the instructions of the manufacturer.
- The manufacturer will not be responsible of the damage caused by an incorrect ground connection. The damage caused by a faulty installation or usage of the unit will not be covered by the warranty.
- In case of doubt, do not start the unit. Contact a qualified and authorised technician.

- Do not touch the unit or the power cord with wet hands.
- Unplug the unit in case of storm and when you are not going to use it for a long time.
- Place the unit in a dry place, away from dust and water splashes.
- Do not pull, twist or fold the power cord excessively.
- Do not wet the unit; do not use chemical products to clean it

- **WARNING:** to prevent electric shock hazards, connect the unit to a proper power source (grounded 230 V, 50/60 Hz).

- The machine features a precision circuit. Prevent fortuitous impacts and stresses that could cause mechanical damage.

- Remove any dirt or water remains from the cord connection.
- Unplug the unit immediately whenever you notice an unusual noise, smell or smoke.
- Do not insert plastic objects into the bowls or any other material that melts at low temperatures.
- Do not use the unit in damp rooms or near inflammable materials (alcohol, solvents,...).

- Do not allow children or non-qualified personnel handling the unit.
- Do not insert metallic rings in your fingers inside the induction cavity.
- Choose a flat and firm surface to place the unit, away from heat or vibration sources.

- Contact the supplier for questions or detailed information about problems with the unit.

INTRODUCTION

The Induction Tool Heater (R-100009) is a simple desktop device to heat metallic spatulas and similar tools used for wax modelling in the Dental Lab. Just insert the head of the tool you want to heat into the hole at the front of the device, and in a few seconds it will reach the optimal working temperature. This way you don't need to use the traditional Bunsen burner and you will prevent its drawbacks: gas tanks, fumes, fire hazard, etc.

FUNCTIONS AND FEATURES

- High temperature in a short time.
- Lower risk of fire and burns.
- Its two electronic sensors provide a higher energetic efficiency.

- Easy to use and ergonomic design.
- Easy to clean (disposable shield)
- Safety: whenever the instrument remains inside the hole for a long time, an error sign will be shown (quick flickering of green and red lights).

INSTALLATION

- Place the unit in a flat, horizontal and stiff surface, away from vibration or heat sources. Make sure there is not any flammable material nearby.
- Connect the unit to a grounded 230 V, 50/60 Hz power source.

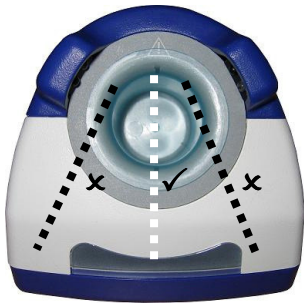
OPERATION

- Connect the power cord and turn the ON/OFF switch on.

- Make sure the green light is on.

- Place the tool in the middle of the induction cavity making sure you do not touch the inner wall.
- The sensors will detect the tool or your hand (it could be difficult if the tool is too thin).

- The red light will light up and the tool will reach a high temperature very quickly.

- As a safety measure, if a tool is kept inside the induction tank for a long time, the heater will stop and will show a light sign (quick flickering of red and green lights).

WARNING!: do not insert metallic rings in your fingers inside the induction cavity.

MAINTENANCE AND CLEANING

To ensure a long life of the unit and an optimal operation, we strongly recommend:

- To clean the wax remains from the induction cavity, the casing and the sensors periodically. To do this you can use a small spatula. Make sure the unit has been previously disconnected from the power source.
- Do not spill melted wax over the outer casing of the unit. Handle the tools with caution.

TECHNICAL SPECIFICATIONS

Voltage:	AC 220~240 V
Frequency:	50/60 Hz
Fuse:	2 A
Power:	100 W
Height:	89 mm
Width:	91 mm
Depth:	209 mm
Weight:	530 gr

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