**Ingredients / Ingredienti / Zusammensetzung /
Composition / Composició**

Whitening gel: Glycerin, Carbamide Peroxide,
Propylene Glycol, Aqua, Carbomer, Sodium Hydroxide,
Disodium EDTA

Activator gel: Aqua, Glycerin, Propylene Glycol, Potassium Nitrate,
Carbomer, Hydroxyapatite, Sodium Hyaluronate, Disodium Phosphate,
Sodium Fluoride, Phenoxyethanol, Sodium Hydroxide

**Final gel after mixing / Gel miscelato /
Endgültiges Gel nach dem Mischen / Gel fini après mélange /
Gel final después de la mezcla**

Glycerin, Aqua, Carbamide Peroxide, Propylene Glycol, Carbomer, Potassium Nitrate, Hydroxyapatite, Sodium Hyaluronate,
Disodium EDTA, Disodium Phosphate, Sodium Fluoride, Sodium Saccharinate, Phenoxyethanol, Sodium Hydroxide

Fast and gentle home whitening treatment

PROFESSIONAL INSTRUCTIONS

BlancOne[®] HOME is a cosmetic whitening treatment to be used at home with individual trays.

BlancOne[®] HOME is available in two versions: NIGHT and DAY according to the patient's preference for a treatment performed during the day or the night.

BlancOne[®] HOME low viscosity composition prevents dehydration, considerably reducing sensitivity, its physiological pH (5.6 – 6.0) is not aggressive towards the enamel guaranteeing the highest effectiveness of the gel and its storage at room temperature.

BlancOne[®] HOME contains, other than potassium nitrate and sodium fluoride, hydroxyapatite, strengthening the enamel and hyaluronic acid to protect gums.

General information

- BlancOne[®] HOME is available in two versions:
 - > NIGHT – with 10% carbamide peroxide (3.5% hydrogen peroxide) to be applied overnight,
 - > DAY – with 16% carbamide peroxide (5.6% hydrogen peroxide) to be applied during the day for 2-3 hours.
- Each BlancOne[®] HOME bag contains a sufficient quantity for a one-week treatment. It's possible to choose between two treatment solutions: **Complete and Light**.
 - > HOME Complete treatment includes delivery of 2 bags for a 2-weeks treatment.
 - > HOME Light treatment, used for maintenance or in combination with in-office treatments, includes delivery of only 1 product bag for a one-week treatment.
- In addition, BlancOne[®] HOME STARTER KIT is also available, ideal when the home treatment is performed for the first time. BlancOne[®] HOME STARTER KIT includes all materials necessary to create the trays and the starting kit to be supplied to the patient (case, instructions, tray holder, shade guide and BlancOne[®] DUETTO toothpaste).

A. Pre-treatment

- Before carrying out the whitening treatment it's important to perform a thorough examination, to make sure that the soft tissues are healthy and to identify possible defective restorations and areas of decay which will have to be treated accordingly.
- Isolate exposed dentin/root surfaces with BlancOne[®] SEAL to prevent sensitivity or mild pain.
- Remove calculus, tartar build-up and superficial stains by means of an accurate dental hygiene session. In case of tissue trauma, wait for complete healing before starting the whitening treatment.
- Take the impression and create the individual trays following the instructions included in the BlancOne[®] HOME STARTER KIT.

B. Treatment

- Choose the day or night BlancOne[®] HOME treatment, and establish the length of the treatment, 1 or 2 weeks.
- Establish the initial shade and record it on the patient's personal card or in the digital one of the BlancOne[®] PRO App together with the pre-treatment picture. Enter the initial shade on the paper colour guide included in the BlancOne[®] HOME STARTER KIT for the patient.
- Inform the patient about the correct procedure for

preparation, application and storage of the whitening gel, correct positioning of the trays once filled, and finally cleaning and storage of the trays.

- Deliver the patient's kit supplied with BlancOne[®] HOME STARTER KIT together with the gel necessary according to the length of the treatment (1 or 2 bags), explain the use of BlancOne[®] DUETTO toothpaste and decide the following check-up appointment.
- During the check-up, carry out an examination to verify the conditions of the oral tissues, establish the final shade and record it on the patient's card or in the BlancOne[®] PRO App together with a picture after the treatment.

Preparation and storage of the whitening gel

BlancOne[®] HOME whitening gel is a two-component material to be prepared before the use by mixing the two components.

- Remove the cap from the syringe with the whitening gel (Whitening) and insert the connector, included in the package, turning clockwise to block it.
- Remove the cap from the syringe with the activator gel (Activator) and connect the syringe to the other side of the connector firmly, turning the syringe clockwise to block it.
- Pushing the piston of one of the two syringes, transfer the content of a syringe into the other.
- After, push the piston of the other syringe to completely transfer the content to the empty one.
- Repeat the last passage 10-15 times to obtain the complete mixing of both gels.
- Completely transfer the gel in a syringe and remove the connector and the empty syringe.
- Connect the application tip to the filled syringe, detach the tip end-cap and close well the syringe turning from the other side the tip end-cap and placing this at the opening of the tip.
- Store the syringe in the fridge for further applications and up to 10 days.

Whitening gel application

The treatment consists in the daily application of the gel by means of the personalised trays.

The length of the treatment can vary on the basis of the clinical needs, treatment goals and possible combination with in-office treatments. Do not exceed the number of applications suggested by the dentist.

Interrupting the treatment for a few days does not compromise the whitening effect once the treatment is resumed.

- Remove the syringe from the fridge and leave it at

- room temperature for 30 minutes before using it.
- Apply the gel in the front (vestibular) part of the trays at about half height from molar to molar.
- Dry the teeth surface with a cloth.
- Wear the trays once filled, remove the gel in excess and leave on for 2-3 hours in case of daily treatment (DAY) or overnight for the night treatment (NIGHT).
- Remove the trays and clean both teeth and trays from any residual of gel using brush and water.
- Dry and store the trays in the specific tray holder box.
- Do not eat or drink for at least 30 minutes.

European Union warnings

- The final product, after mixing the two components, contains:
 - NIGHT 3.5% hydrogen peroxide as 10% carbamide peroxide
 - DAY 5.6% hydrogen peroxide as 16% carbamide peroxide
- Do not use on patients under 18 years of age.
- It can be solely sold to dental practitioners.
- For each treatment cycle, the first use must be carried out in office or under the supervision of a dentist, where the same level of safety is guaranteed. Only after this it can be made available to the patient to complete the treatment cycle.
- It contains sodium fluoride (500 ppm).

Precautions

- Keep away from children.
- It contains peroxide and fluoride, in case of ingestion in large quantities it can be dangerous. In this case consult a doctor.
- Avoid contact with the eyes. In case of contact immediately rinse with abundant water.
- Oral and gums hygiene must be good before starting the treatment. Before the whitening treatment it's necessary to carry out a dental examination.
- If the patient is subject to allergies or sensitisation to one of the product components, the treatment must not be carried out.
- Not suitable for pregnant women. If breastfeeding, consult a doctor.
- Some patients can feel sensitivity or gingival irritation. These are generally mild and temporary effects. If the inconvenience persists, interrupt the treatment and make an appointment to assess the problem.
- The treatment does not whiten crowns, inlays, fillings, non-vital teeth.

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