EN	DIRECTIONS FOR USE	3
FR	PROTOCOLE D'UTILISATION	6
IT	ISTRUZIONI PER L'USO	9
DE	GEBRAUCHSANWEISUNG	12
ES	PROTOCOLO DE USO	15
PT	INSTRUÇÕES DE UTILIZAÇÃO	18
NL	GEBRUIKSAANWIJZING	21
SV	BRUKSANVISNING	24
DA	BRUGSANVISNING.....	27
NO	BRUKSANVISNING	30
LT	NAUDOJIMO INSTRUKCIJOS	33
LV	LIETOŠANAS NORĀDĪJUMI	36
ET	KASUTUSJUHISED	39
CS	NÁVOD K POUŽITÍ.....	42
SK	NÁVOD NA POUŽÍVANIE	45
HU	HASZNÁLATI UTASÍTÁS	48
RO	INSTRUCȚIUNI DE UTILIZARE	51
SR	UPUTSTVO ZA UPOTREBU	54
BS	UPUTSTVO ZA UPOTREBU	57
CNR	UPUTSTVO ZA UPOTREBU	60
SQ	UDHËZIME PËR PËRDORIMIN	63
EL	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ	66
PT-BR	INSTRUÇÕES DE USO.....	69
KO	사용 지침	72

Gutta-Percha Points

FOR DENTAL USE ONLY

DIRECTIONS FOR USE

REF. A175X, A241X, PULG

0) INDICATIONS FOR USE

Application field:

Endodontic obturators.

Intended purpose:

Gutta-percha points are intended to fill the shaped, cleaned and irrigated root canal space.

Indication for use:

Gutta-percha points are indicated for treatment of endodontic disease.

1) CONTRAINDICATIONS

None.

2) DELIVERY FORMS

Not applicable.

3) COMPOSITION

Components:

- Zinc Oxide
- Titanium Dioxide
- Synthetic gutta-percha
- Barium Sulfate
- Coloring Agents
- Inert fillers

4) COMPATIBLE DEVICES

Gutta-percha points are used in combination with:

- Spreaders and Pluggers. These instruments are used to assure the gutta-percha point is distributed throughout the entire root canal system.
- Calamus® and Gutta-Smart™. They are heating devices that are used to extrude gutta-percha material into the root canal system.
- Gutta-Condensor. Gutta-Condensor is a thermal condenser used to soften and fuse gutta-percha points together.
- AH Plus® Sealer. AH Plus is used to coat root canal walls and remains at the interface between the root canal and the gutta-percha.

5) WARNINGS

Single-use products shall not be re-processed nor reused. Reusing these products increases the risk of cross-contamination and/or breakage.

6) PRECAUTIONS

General:

- Safety and effectiveness of use have not been established in pregnant or breastfeeding women or in children.
- Inspect the packaging and product before use: if the packaging or the product is damaged, do not use the product and discard it.
- Products shall be disposed of according to local regulations for the safe disposal of sharp and contaminated devices.
- Not to be sterilized with heat.
- Do not use disinfecting solutions containing Phenol or any products which are not compatible with gutta-percha.

7) ADVERSE REACTIONS

In the present technical state, no adverse reaction has been reported so far.

8) STORAGE CONDITIONS

- Keep away from sunlight and heat.
- Maintain under room temperature (< 30°C (86°F)).
- Store in a dry area.

9) STEP-BY-STEP INSTRUCTIONS

The gutta-percha points shall only be used in a clinical or hospital environment, by qualified dental professionals. The described gutta-percha points are part of a complete system and should only be used in combination with the corresponding shaping files and paper points. They are identifiable with their system brand, size name and color code. After shaping and cleaning the root canal, obturation technique must be set by the professional. This procedure includes the use of gutta-percha points and sealer. Gutta-percha points shall be selected from the same shaping system and shall correspond to color code and shaping file size of the last instrument used during canal preparation. The points are to be disinfected prior to use. A heated instrument can be used to soften the material and ensure perfect sealing and dimensional adaptation of the root canal. In the use of a heated instrument, follow the IFU of said instrument. Use an appropriate tool to cut the gutta-percha points, including color coding ring after the insertion of the gutta-percha point in the root canal. The colored ring shall not remain in the root canal after insertion.

10) DISINFECTION

Immerse the gutta-percha points in 5.25% NaOCl for 1 to 5 minutes at ambient temperature. Remove all the bubbles from the surface of the gutta-percha point. Gently wipe the gutta-percha point with a sterile gauze moistened with 70% isopropyl alcohol. Let the gutta-percha points air dry.

Do not use disinfecting solutions containing Phenol or any products which are not compatible with gutta-percha.

11) SUMMARY OF SAFETY AND CLINICAL PERFORMANCE

Information is available in the European database on medical devices (EUDAMED), where it is linked to the Basic UDI-DI.

<https://ec.europa.eu/tools/eudamed>

The Basic UDI number is: TBD

12) LOT NUMBER AND EXPIRATION DATE

Do not use after expiration date.

13) SERIOUS INCIDENT REPORTING

Any serious incident in relation to the product should be reported to the manufacturer and competent authority according to regulations.

Disclaimer:

Users are solely responsible for any deviation from these instructions, and/or the use of alternative methods for processing. Dentsply Sirona accepts no liability for damage, injury, or any legal responsibility incurred directly or indirectly by the user due to a deviation from the instructions for use set forth below. The user shall observe safe and lawful practices including, but not limited to, those set forth in this document.

Authorised Representative

DENTSPLY DeTrey GmbH
De-Trey-Straße 1
D-78467 Konstanz
Germany

CE
2797

Manufacturer

DENTSPLY Tulsa Dental Specialties
608 Rolling Hills Drive
Johnson City, TN 37604, USA
www.dentsplysirona.com/ifu

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