

Meta P&Bond 5th Generation Bobbing System

This bonding agent is one step for priming and bonding

Meta P&Bond adhesive system is wet bond adhesive system that is activated by visible light curing. This bonding agent is one step for priming and bonding

INDICATIONS

- Bonding for direct composite restoration
- Bonding for Porcelain, composite, metal repair, set amalgam, post
- Bonding for porcelain veneer
- Bonding for amalgam and indirect bonding procedures with Metacem adhesive resin cement

INSTRUCTIONS

- ① **Isolation:** Rubber dam is the preferred method for isolation.
- ② **Cavity preparation:** prepare cavity with minimal tooth reduction.
- ③ **Tooth conditioning /Dentin pretreatment:** Apply 37% Meta Etchant gel to available enamel/dentin for 15 seconds.
- ④ **Rinse and Blot dry:** Remove gel with aspirator tube and rinse for 20 seconds. Blot dry conditioned area with a moist cotton pellet. Blot dry to keep moist, do not rub.
- ⑤ **Bonding:** Apply a double coat of **Meta P&Bond** by using microbrush. Gently air dry for 5 seconds. Light cure for 10 seconds.
- ⑥ Restoration or cementation.

CONTRAINDICATION

- **Meta P&Bond** should not be used if patients are known to be allergic to and of the ingredients

SIDE EFFECTS

- None known to date

CAUTIONS

- Avoid contact of **Meta P&Bond** with the skin / mucous membrane, or eyes.
- Unpolymerized **Meta P&Bond** may cause slightly, reversible irritation or lead to a general sensitisation againsts methacrylates.
- Keep away from child.
- For dental use only.
- Do not use **Meta P&Bond** after the date of expiration.



INSTRUCTION FOR USE

STORAGE

- Close bottle immediately after use to minimize evaporation
- Exposure to light may be lead to premature polymerization
- Store at 18°C~24°C(64°F~75°F)

COMPONENT

- **Meta P&Bond** 5ml/g in bottle
- Microbrush
- Instruction for use

SHELF LIFE

- 2 years from the manufactured date.

SYMBOLS



Use By



Batch Code



CE-Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC



Attention, See instruction for use



Manufacturer

Made in Korea

Manufactured by
META BIOMED CO.,LTD.

634, Yeonje-ri, Gang-oe-myeon, Cheong-won-gun
Chungcheongbuk-do, Korea
Tel: +82-43-216-0433
Fax: +82-43-217-1988
E-mail: info@meta-biomed.com
Web site: <http://www.meta-biomed.com>